



STRATOS MASK

THE GENERIC HYGIENIC
PROTECTION MASK FOR DAILY USE





STRATOS MASK

STRATOS MASK IS THE GENERIC HYGIENIC PROTECTION MASK THAT LA SPORTIVA HAS DESIGNED FOR DAILY USE IN THE COEXISTENCE PHASE WITH THE COVID-19 VIRUS.

To limit the spread of the Coronavirus in fact, each of us must wear a mask in order to avoid inadvertently passing the virus on to other people. Stratos Mask ergonomically wraps the wearer's face, avoiding the dispersion of saliva droplets that could potentially contain the virus.

Attention has been paid to **comfort, stability, breathability** and **safety**.

TECH INFO

Stratos Mask is the **generic hygienic protection mask** made of 3D mesh fabric with an internal, certified interchangeable anti-bacterial filter. The mask is washable, reusable and a comfortable to wear product thanks to perfect ergonomics designed to wrap around the face in a safe and secure way.

The La Sportiva **Stratos Mask** is ideal for daily use thanks to the enhanced comfort and ease of replacing the filter.

- 1 **Ergonomic shape.**
- 2 **3D-Mesh fabric** for high breathability with ViralOff by Polygiene anti-bacterial and anti-viral treatment.
- 3 **Flexible support frame** for maximum comfort and breathability.
- 4 **Sensitive Comfort Layer™**: internal Lycra® gaiters on the nose and chin for maximum comfort.
- 5 **Adjustable elastic straps** for precise and safe fit.
- 6 **Internal filter** in polypropylene melt brown **certified** for bacterial filtration and interchangeable with maximum ease.
- 7 **Security-hole** for checking the correct position of the filter.



20



S (kid) / M / L



Warning: despite having a standard's certified filter, Stratos Mask is not to be considered a medical device.





ViralOff by Polygiene antiviral and antibacterial treatment

All components of the mask are treated with **ViralOff by Polygiene** which guarantees a reduction of viruses and bacteria by 99% in less than two hours (ISO 18184:2019). This allows to have a mask that is always clean every time you wear it. Thanks to the ViralOff treatment, it is not necessary to wash the mask after each use.



Interchangeable polypropylene filter

A single-layer filter, certified for bacterial filtration is inserted inside the Stratos Mask. It is easily replaceable thanks to the comfortable straps. It allows to **filter about 99% of bacteria and viruses** emitted by the mouth, exactly as required for surgical masks according to EN ISO 14683:2019 standards.



When to replace the filter

It is advisable to replace the filter daily, and even more frequently in the case of intense use such as a sports activity or use for several consecutive hours at work. In case of heavy use La Sportiva recommends a replacement approximately every **4 hours**. The mask is a lasting accessory: it can be washed several times. It should be noted that the ViralOff anti-bacterial treatment will gradually decrease over time due to normal wear and tear of materials after washing.



Warning: it is recommended to always wash your hands before putting the mask on and off.





MATERIAL CHARACTERISTICS



HYGIENIC

The 3D Mesh fabric is certified and produced in Europe. It is a hygienic and anti-allergenic fabric.



SUSTAINABLE

The use of a washable and reusable mask compared to the single-use ones contributes to the protection of the environment, the smaller surface of the filter compared to that needed to produce sanitary masks also further limits the waste of material.



RESISTANT

The fabric of the mask frame is abrasion resistant and long lasting even after several washes.



WASHABLE

Once the filter has been removed, the mask can be completely washed by hand or in the washing machine at 40 degrees.



BREATHABLE

The 3D Mesh fabric and the three dimensional ergonomic shape make the product very breathable.



COMFORTABLE

The soft, internal Lycra® gaiters allow for maximum comfort.



ADAPTABLE

It is possible to adjust its ergonomics by raising the chin cover and lowering the nose cover to increase breathability.




ANTIVIRAL TREATMENT

The 3D mesh and the internal Lycra® inserts use the ViralOff by Polygiene treatment, which makes them both antibacterial and antiviral.




PATENT PENDING



INSTRUCTIONS FOR USE

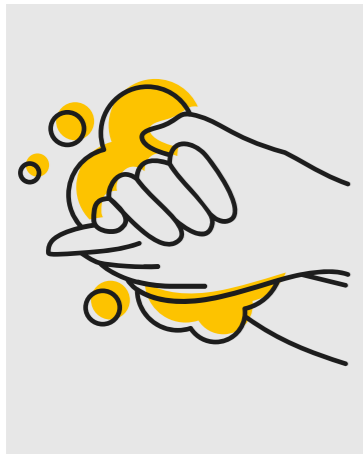

- 1 Carefully wash your hands and face before handling and wearing the mask.

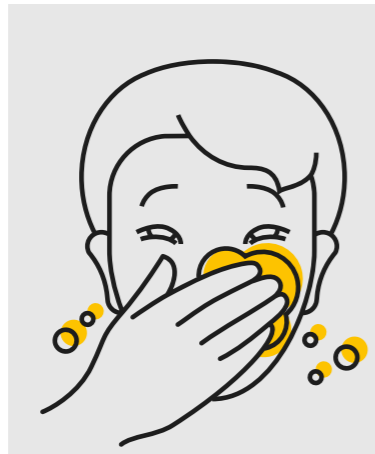


- 2 Wear it by placing it on your face and passing the laces behind your head.

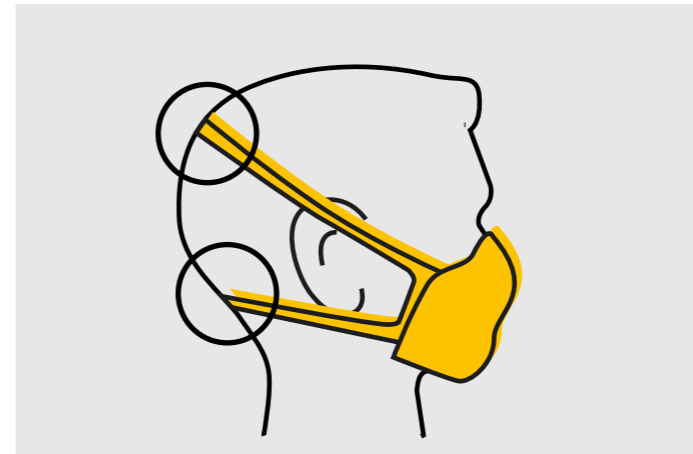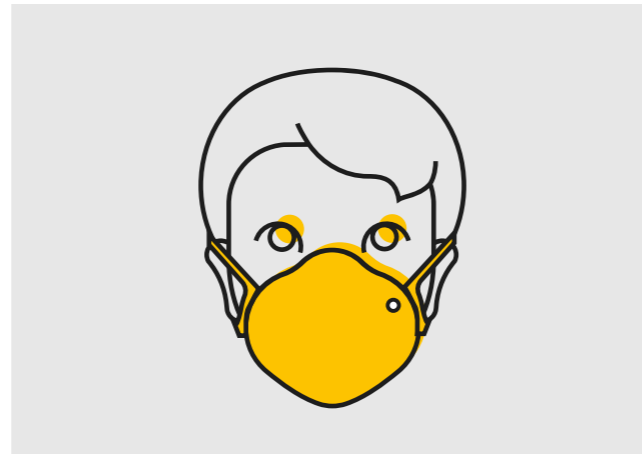


- 3 Adjust the tension of the laces so that it adheres correctly to the face without empty spaces.




HOW TO REMOVE AND CHANGE THE FILTER

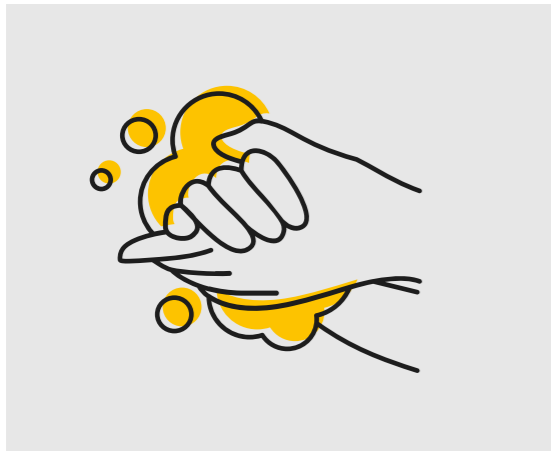


- 1 Wash your hands thoroughly before removing the filter from the package.

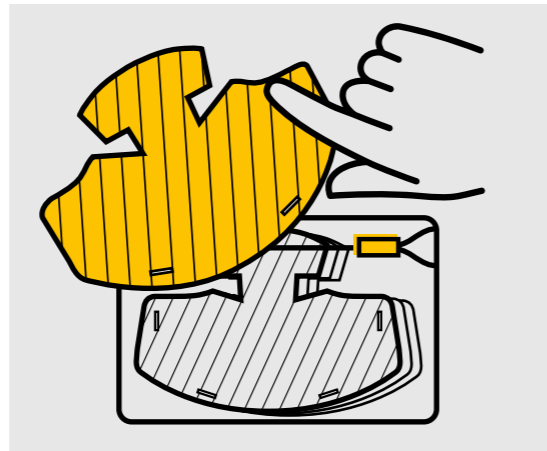

- 2 Close the package immediately after removing one of the 30 single-use filters.

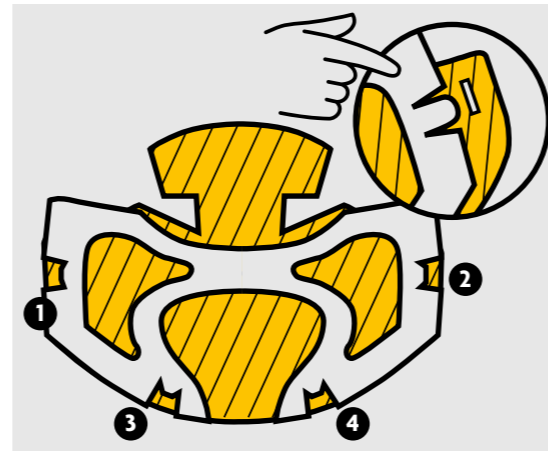

- 3 Place the filter on the internal frame following the instruction order in the picture.

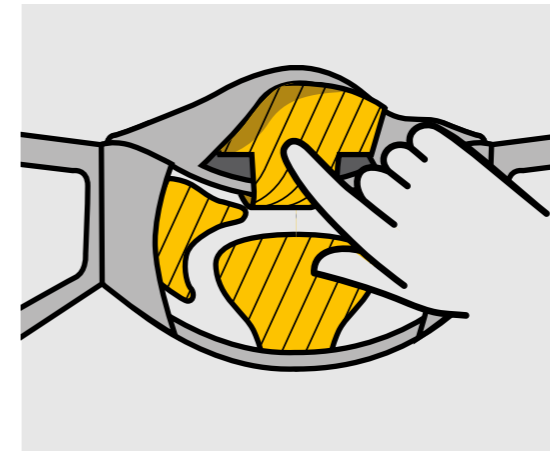

- 4 Insert the frame into the mask, keeping the filter in contact with the mask.

Insert the upper part of the filter (corresponding to the nosepiece), in the Lycra® pocket.



- 5 To remove, proceed in reverse order: lift the tabs and remove the filter. The filter must then be thrown in the **general waste**.

Wash your hands thoroughly at the end of the process.

WASHABLE AND REUSABLE



The **STRATOS MASK** can be hand or machine washed at 40 °C, after carefully removing the filter and the elastic laces. The possibility of washing makes the mask long-lasting and contributes to the reduction of waste compared to disposable masks, however it is advisable that even after washing, Stratos Mask is used for exclusive personal use only, avoiding passing it on to other people, even family members.

Do not dry clean.



STRATOS MASK



LA SPORTIVA